GERÄTEDOKUMENTATION

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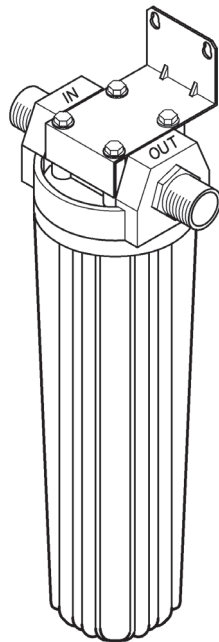
Vollentsalzungspatrone

DOC 85635 / 06

- de Deutsch
- en English
- fr Français
- it Italiano
- es Español
- ru Русский


Vollentsalzungspatrone

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
Vollentsalzungspatrone

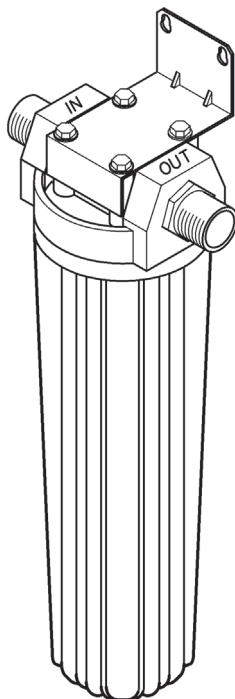de Deutsch

Vollentsalzungspatrone

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
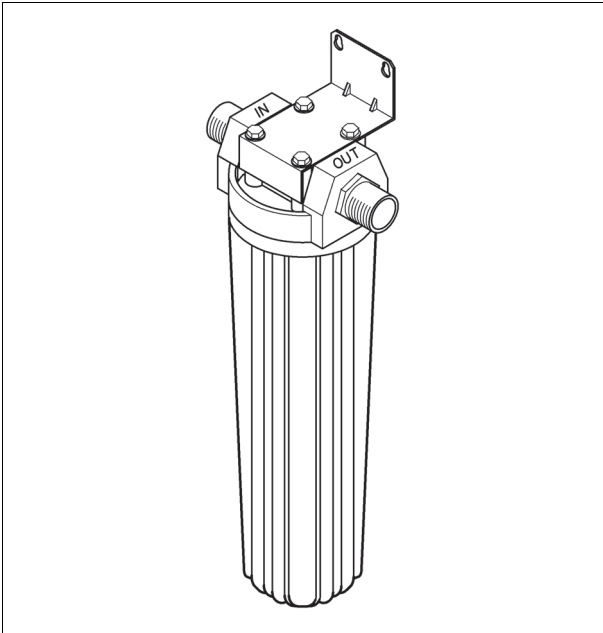
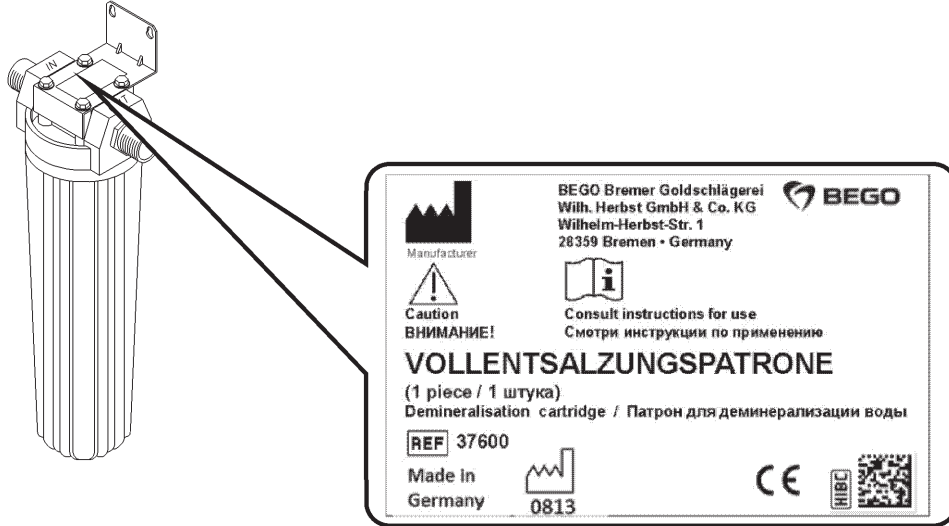
Translation of the original instructions

Full demineralisation cartridge

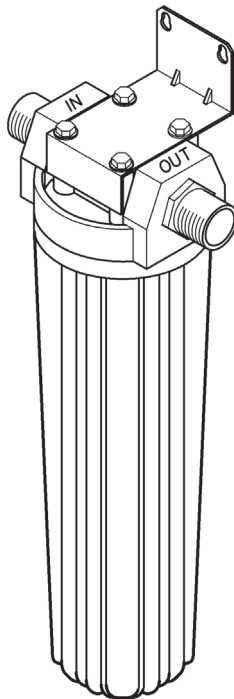en English

Full demineralisation cartridge

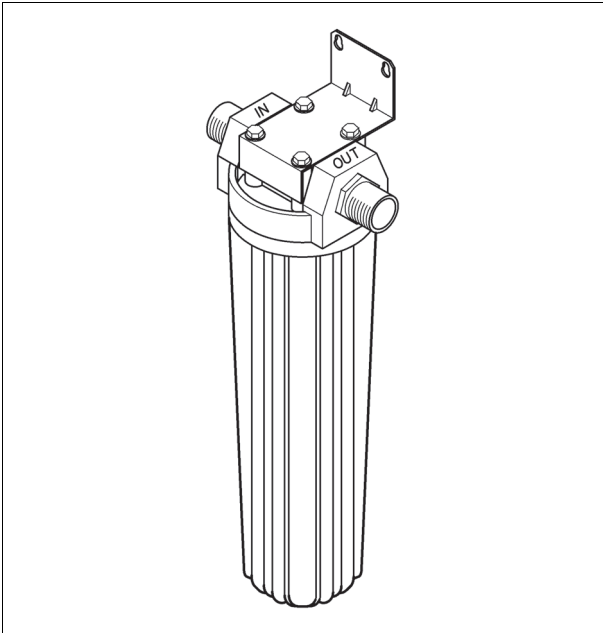This unit documentation is part of the unit and must be enclosed when selling or passing on the unit.

- The unit has been designed solely for use in dental laboratories and comparable establishments for research, commercial and training purposes. The unit must only be operated by dental professionals, or they must provide supervision if trainees or other persons operate the unit.
- The operating instructions must be read and understood before the device is used. This particularly applies to the **Safety information**. The warranty is void in cases of damage which is caused by non-observance of the operating instructions. We will also not accept liability for any resulting consequential damages.
- Symbols used

This symbol indicates very important information. Failure to comply with it may result in personal injury.

Table of contents

General Information	13
Safety Instructions.....	15
Unit description	16
Scope of delivery and Accessories	16
Technical Data.....	17
Installation	17
Operation	17
Service and Maintenance.....	18
Elimination of malfunctions	19
Signs, labels and stickers.....	20

General Information

Importance of the operating Instructions

These operating instructions cover all the information that is necessary according to the relevant regulations for the safe operation of the device described herein.

The operating instructions are a part of the machine. The operating instructions should therefore

- always be kept at hand near the device until the device is disposed of,
- and should be passed on with the device when it is purchased, sold, or lent.

Contact the manufacturer if you do not understand something in the operating instructions.

We welcome any suggestions or criticism. Please feel free to notify us. Your effort will help us make the operating instructions more user-friendly and respond more effectively to your wishes and needs.

Target group

This document is directed toward everyone who works with this device or performs service tasks that are described in this document.

Contact information

BEGO Bremer Goldschlägerei Wilh. Herbst GmbH & Co. KG
Wilhelm-Herbst-Str. 1
28359 Bremen • Germany
Phone: +49 421 2028-0
Service-Hotline Phone: +49 421 2028-270
www.bego.com

© Copyright by BEGO Bremer Goldschlägerei Wilh. Herbst GmbH & Co. KG

This user guide is protected by copyright. All rights reserved.

No part of this document may be reproduced without the express written permission of BEGO Bremer Goldschlägerei Wilh. Herbst GmbH & Co. KG!

Conventions

These operating instructions contain references to residual hazards, important user tips and handling instructions that are identified with the following symbols and words.

DANGER

This reference identifies hazards that can cause serious bodily injury or death if the relevant hazard instructions are not followed or not followed properly.

WARNING

This reference identifies hazards that can cause bodily injury or property damage if the relevant hazard instructions are not followed or not followed properly.

ATTENTION

This reference identifies only those hazards that are potential damaging to property and the environment.

NOTE

This symbol identifies user tips and particularly useful information. It helps you optimally exploit all the functions of your machine.

1. Sequential actions are described in sequentially numbered paragraphs.
- Cross-references are identified with this symbol.

Warranty and Liability

Our general terms and conditions of sale and delivery apply. These are made available to the customer on completion of the contract, at the latest. Warranty and liability claims in the event of physical injury or damage to property are invalid if they are caused by one or more of the following:

- Improper use of the device
- Improper installation, putting into operation, operating and maintenance of the device;
- Operating the device with safety and protective facilities that are defective, improperly installed or inoperative
- Failure to heed the information in this manual concerning the transport, storage, installation, putting into operation, operation and service and maintenance of this device;
- Unauthorized structural modifications to this device;
- Inadequate monitoring/checking of parts of the device that are subject to wear;
- Improperly performed repairs;
- Catastrophes caused by alien elements or force majeure.

Safety Instructions

The full demineralisation cartridge removes salts from tap water for the Triton SLA steam cleaner. Any other use is considered to be improper. We shall not be liable for any damage resulting from improper use.

The safety instructions must be followed in order to prevent personal injury and damage to the machine. We shall not be liable for damage due to failure to comply with the safety instructions.

General Safety Instructions

Place of use - The unit has been designed solely for use in dental laboratories and comparable establishments for research, commercial and training purposes.

The unit may only be installed on an adequately stable surface.

The unit must only be used in dry rooms.

Operator and operating personnel - The unit must only be operated by dental professionals, or they must provide supervision if trainees or other persons operate the unit.

Wear the protective clothing prescribed for use in dental laboratories and observe the rules of conduct.

The unit may only be operated by staff members who are familiar with and follow the instructions of this operating manual.

The operating manual must be easily accessible when the machine is operated. Observe the national accident prevention regulations in addition to this operating manual.

Handling the machine - Modifications of the machine are not permitted.

Always keep signs and stickers such that they are easy to read. They must not be removed.

Check the machine and leads for damage regularly. The machine must not be operated if it has any defects that may result in risks for the employees or third persons.

Protect the leads against heat, oil and sharp edges. The leads must not be used to carry the machine or to pull out the mains plug.

Repair work - Repairs may be carried out solely by BEGO customer service or by persons authorised by BEGO customer service.

Only BEGO replacement and wearing parts may be used.

Care and maintenance. The unit has been designed for a service life of 10 years from the date of manufacture. No liability is accepted for damages arising from operation of the unit after this period.

Switch off the machine and pull out the mains plug before performing any care, cleaning or maintenance work.

Only use dry or slightly moist cloths for cleaning. Do not spray the machine with water or immerse it in water.


Special safety instructions regarding this machine

⚠ WARNING	
	<p>Unit is under pressure!</p> <p>The casing of the cartridge is pressurized. Relieve water pressure on it before opening.</p>

NOTE	
<ul style="list-style-type: none"> The sensor of the waterstop valve may switch the water supply off in the event of a leak. It is not necessary to dispose of the insert or the Durox separately. 	

Unit description

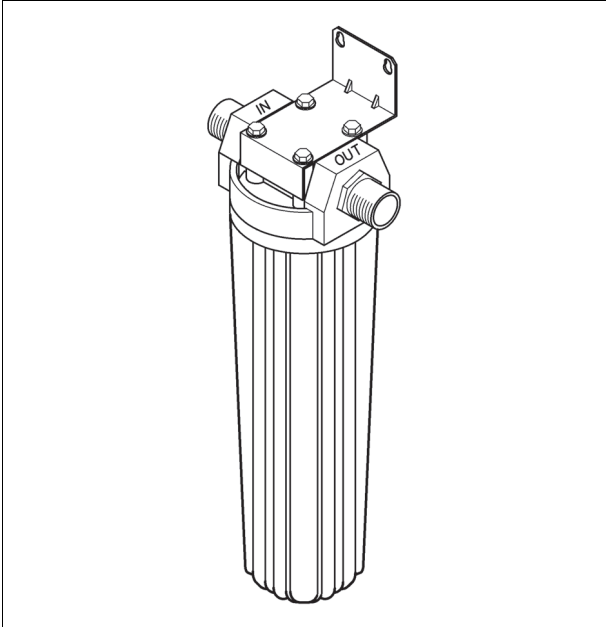Full demineralisation cartridge

The full demineralisation cartridge contains an insert with an ion-exchanger resin. This removes all of the salts from the tap water flowing through, particularly the calcium and magnesium that cause hardness. The resin can only absorb a certain amount of salt. Once its capacity is exhausted the cartridge's insert is to be replaced with a new one.

The insert can be emptied, cleaned and refilled with Durox replacement resin quite simply.

Scope of delivery and Accessories

Scope of delivery

Full demineralisation cartridge REF 37600

with:

- Mounting angle
- Fixings
- 2 Ion-exchanger inserts
- Ring spanner
- Operating instructions

Accessories

	REF
Durox non-reusable ion-exchanger resin, 6 l	52121
Insert for full demineralisation cartridge with seal ring:	
1 piece (new).....	12260
1 piece (recycled).....	12260 T
2 pieces (new).....	37602
2 piece (recycled).....	37602 T
Ring spanner.....	11044

Technical Data

Height.....	625 mm	Maximum temperature of water supply	30 °C
Width	200 mm	Maximum flow rate	120 l/h (at 4 bar)
Depth	150 mm	Capacity	250 l at a total salt content of 100 mg/l
Minimum installation height required	660 mm		(Water hardness 3)
Water supply	3/4", 4...6 bar		
	(0.4-0.6 MPa)		

Installation

1. Mount full demineralisation cartridge vertically in the vicinity of the steam cleaner after marking and drilling 8 mm diameter holes for the plugs, using the holes in the mounting angle as a template. The casing must have a **minimum distance to the ground of at least 30 mm** to enable it to be opened to change the insert.
2. Mount the cartridge with the screws provided.
3. Connect the hose from the supply of untreated tap water to the inlet. Do not forget to fit seal! The pressure of the tap water must be between 4 and 6 bar.
4. Connect soft water hose leading to the steam cleaner to the outlet using a seal.
5. Connect waterstop valve, filter and conductivity meter section in accordance with the operating instructions for the cleaner.
6. Remove softened water hose from the steam cleaner and hold in a bucket.
7. Turn the tap on to fill the cartridge. Allow 2 to 3 litres of water to flow into the bucket before reconnecting the softened water hose to the unit.

Operation

1. Turn tap on.
2. Switch on steam cleaner. The steam cleaners are then automatically filled with softened water.
3. When the "µS" indicator lamp on the steam cleaner comes on the capacity of the cartridge is exhausted and the insert must be changed.
4. Once work is complete switch steam cleaner off and **turn tap off**.

Service and Maintenance

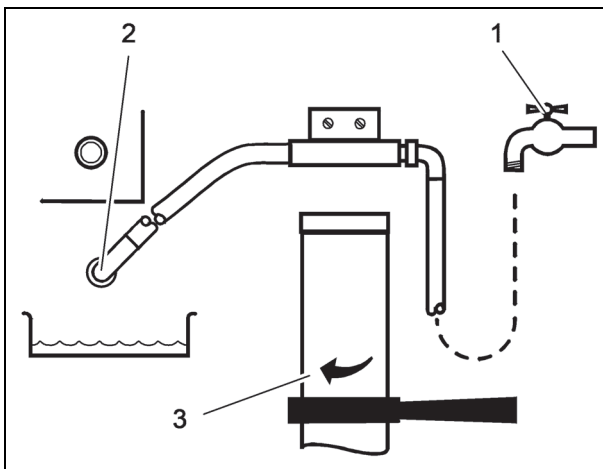
⚠ WARNING


Unit is under pressure!

- The casing of the cartridge is pressurized. Relieve water pressure on it before opening.
- Switch steam cleaner off and allow pressure to drop to zero!

Replacing the full demineralisation cartridge insert

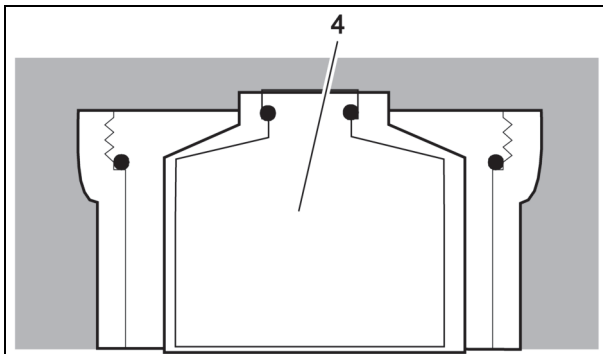
1. Turn tap (1) off.
2. Unscrew hose (2) and allow water to drain.
3. Unscrew casing (3) using ring spanner.


4. Replace insert (4).
Ensure the seals are correctly positioned.

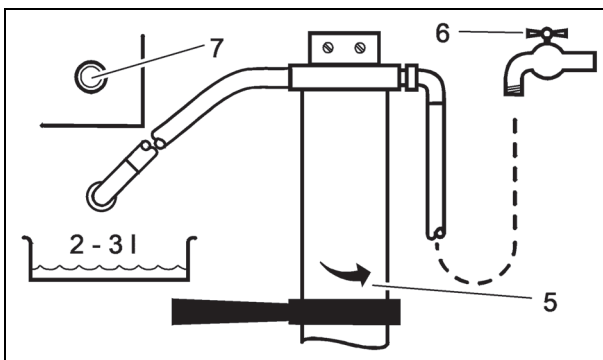- 1 Insert (new) REF 12260
- 2 Inserts (new) REF 37602

5. Screw casing (5) on again tightly using ring spanner.
6. Turn tap (6) on, allow 2 or 3 litres of water to flow through, turn tap off.
7. Screw hose (7) on tightly.

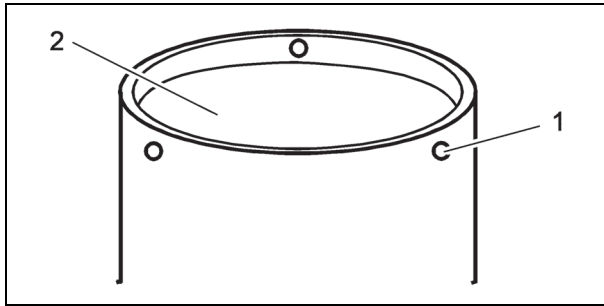Refilling the insert

You can also fill the inserts with Durox yourself if you want to:

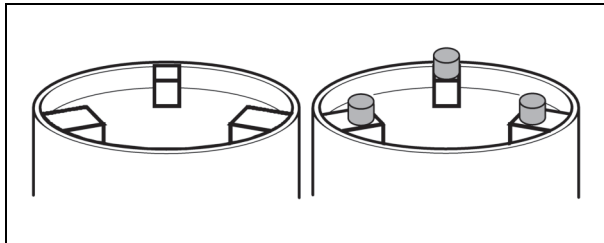
Durox non-reusable ion-exchanger resin, 6 l 52121

1. Pull rivets (1) out.
2. Remove bottom section (2).

NOTE

Dispose of spent resin in the same way as synthetic resin.

3. Wash insert thoroughly under running water.
4. Pour in about 2.3 litres of Durox replacement resin and compact by placing on a vibrator. Fill in Durox up to about 5 mm below the rivet-holes.

⚠ ATTENTION

Property damage!

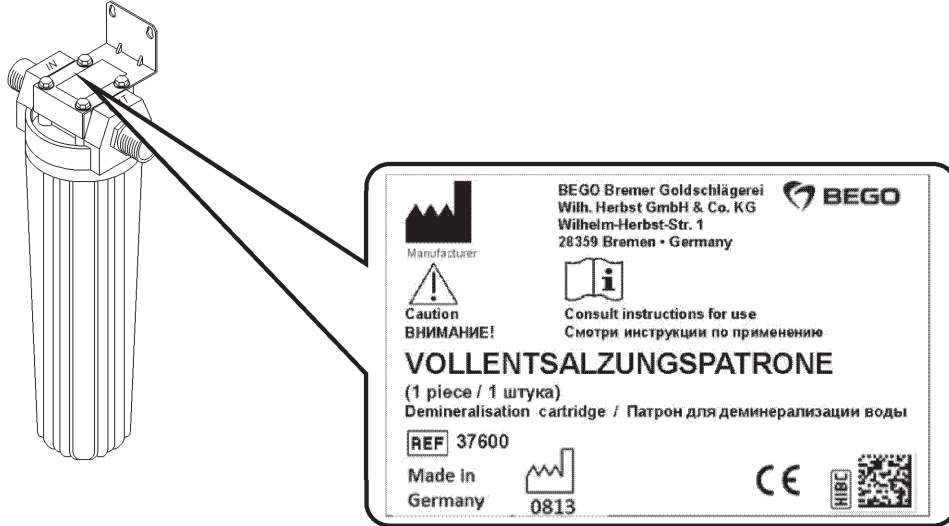
There are different bottoms (see figure). To avoid damage when installing the inserts, do not interchange the bottoms!

5. Refit bottom section with rivets.
6. Fit seal ring (see “Replacing the full demineralisation cartridge insert”, page 18) and store insert so that it is protected from dust until used.

Elimination of malfunctions

- The casing must be mounted vertically, do not just lay it down (→ page 17).
- Once the capacity of the cartridge is exhausted, the steam cleaners switch the water supply off. Change insert (→ page 18).
- Do not use any metal pipes between softened water outlet and steam cleaner.
- Check that all seals are in the correct position.
- The flow is restricted to 120 l/h by an orifice.
- Filter in the softened water hose may need to be cleaned.

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